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en ASSEMBLY INSTRUCTIONS

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en Information for the user

The device is easy to use and presents no difficulties to use even for people with visual impairments or with poor cognition (weak perception).

Possible events that could prevent the ease of use, are excluded.

The surface of the device is not affected by environmental conditions.

The device must be installed in the shower area only.

Do not use the seat outdoors or in saunas.

Cleaning should be done with a damp cloth using a mild soap or a mild detergent. It is strongly recommended not to use cleaners that contain acid components, abrasives, chlorine, solvents such as nail polish remover, acetone, trichloroethylene or derivatives that may damage the surface.

Do not use hot air sources, such as hair dryers, etc. to dry the surface.

It is recommended at least once a week disinfect the surface with a suitable product, noting the warnings and precautions for use recommended by the manufacturer.

Tighten both nuts as shown in figure 6 until the seat remains in balance. If after a certain time there is a free rotation excessively rapid and it is necessary to retighten the nuts as shown in figure 6. The adjustment of the rotation of the seat must be checked at least every three months.

For further information, please visit our website or contact our dealer nearest to you.